



Smart Stage Instruction Card

Smart Stage increases the accuracy of your AXIS readings and provides a Pupillary Distance (PD) measurement. Simply attach Smart Stage to the eyeglasses before running the Netrometer.

To use the Smart Stage

- 1 With the EyeNetra label facing up, squeeze the grips together to open the Smart Stage.
- 2 Orient the temples of the eyeglasses towards you, and place the Smart Stage over the spectacles, centering it in the middle of the bridge.
- 3 Release the grips to clamp the spectacles, ensuring that the Smart Stage is centered.
- 4 Select the Smart Stage option of Netrometer.
- 5 During lensometry, hold the Smart Stage so it lies flat. Refrain from using the trigger.
- 6 Follow the onscreen instructions to measure the lenses, center the dot as best as you can.

